


167643 Blodlansett 200-pack

Color	Dimensions	Blood Volume
	Needle: 30 G x 1.5 mm depth	Low Blood Flow Single Drop
	Needle: 21 G x 1.8 mm depth	Medium Blood Flow
	Blade: 1.5 mm width x 2.0 mm depth	High Blood Flow

Instructions For Use

- 1 Identify the desired puncture site (shaded area) and make sure it is properly cleansed according to your facility's established procedures.

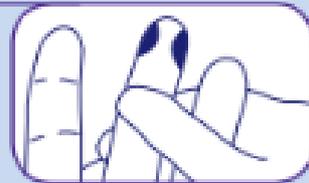


- 2 Twist off tab to break the seal and discard.



- 3 Position safety lancet firmly against puncture site as illustrated. Hold safety lancet between fingers.



- 4 To activate, press safety lancet firmly against the puncture site. Do not remove the device from the site until an audible click is heard.

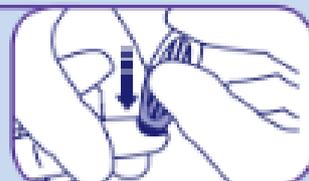

- 5 Discard used safety lancet into a sharps container.

